



Pregnancy



- 1. Flush the wound with a physiological solution while cleaning the wound routinely.**

- ✓ Super-oxidised water
Microdacyn[®], Oculus
- ✓ Saline or Ringer's solution
several manufactures
- ✓ Polyhexanide, e.g.:
Serasept[®], Lavasorb[®], Lavanid[®]
- ✓ Octenidine dihydrochloride, e.g.:
Octenisept[®], Octenilin[®]
(rinse afterwards with saline or
Ringer's solution)



- 2. Spray Granulox uniformly from about 5 -10 cm. To avoid surface drying: Clean spray head after application with a cloth soaked in alcohol. Clogged spray head? Remove and replace.**



3. Cover the wound with a breathable / non-occlusive wound dressing.

✓ **Hydropolymer dressings/foam dressings, e.g.:**

Tegaderm™Foam, 3M; Mepilex®, Mölnlycke; Draco Foam, DRACO®; Allewyn®, Smith&Nephew; Cutimed® Sorbact®, BSN; UrgoCell, Urgo; Hydro Tac®, Hartmann; Suprasorb®P, Lohmann&Rauscher; Biatain®, Coloplast

Spacer grids, e.g.:

Adaptic Touch®, UrgoTül, Urgo; Mepitel® One, Mölnlycke

Superabsorbents/hydrofibres, e.g.:

Aquacel®, ConvaTec; Vliwasorb®, Lohmann&Rauscher; Sorbion Sachet S, Sorbion

✓ **Hydrogels, e.g.:**

NU-GEL®, Systagenix; UrgoHydrogel, Urgo; Intrasite Gel, Smith&Nephew; Hydrosorb® Gel, Hartmann; Purilon®, Coloplast

Film dressings, e.g.:

Tegaderm™ Film, 3M; OPSITE® Flexifix, Smith&Nephew

Alginates, e.g.:

Sorbagon®, Hartmann; Trionic®, Systagenix; Melgisorb®, Mölnlycke; Cutimed®Alginate, BSN; Kaltostat®, ConvaTec; Biatain® Alginate, Coloplast; Suprasorb® A, Lohmann&Rauscher

✗ **Hydrocolloid dressings/Occlusive film dressings/VAC-Systems**

- Storage:**
- Longer period: fridge, 2°C – 8°C
 - Days of treatment: room temperature, max. 25°C.
- When used daily: up to 6 weeks.